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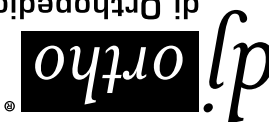
APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully. Correct application is vital to the proper functioning of the brace.

INDICATIONS: • Patients with symptoms, physical findings or radiographic evidence of Unicompartmental Degenerative Joint Disease. This product was designed to complement the variety of medical treatments common to the above afflictions. The frequency and duration of use should be determined by your prescribing Healthcare professional.

A While sitting on the edge of a chair, step through the brace. Flexing the knee to 45°, position the brace as shown in figure 1, centering the hinge about 1" (2.54 cm) above the top of the kneecap and slightly past midline of the leg. Make sure the brace is not twisted on the leg.

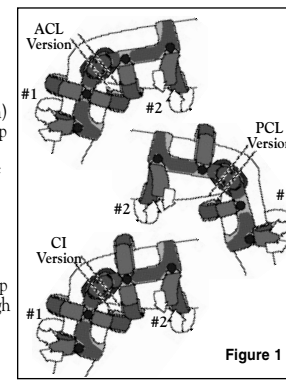
Figure 1

B Secure the bottom calf strap first (#1), then the top thigh strap (#2).

C Keeping the knee bent at 45 degrees, push both condyles (hinges) toward the back of the leg, as shown in figure 2.

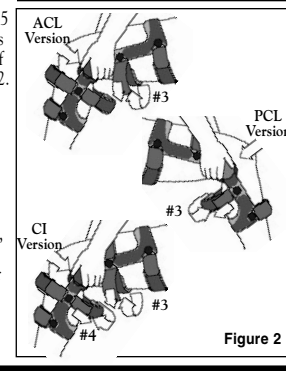
Figure 2

D Maintaining the position of the hinges: for ACL version secure lower thigh strap (#3), for PCL version secure upper calf strap (#3), for CI version secure lower thigh strap (#3) then upper calf strap (#4) on the back of the leg.

E For ACL version, draw the hinges forward slightly before fastening the front tibial (shin) strap (#4). For PCL version draw the hinges forward slightly before fastening the front thigh strap (#4). For CI version draw the hinges forward slightly before fastening the front tibial (shin) (#5) and the front thigh strap (#6). Do not pull the hinges beyond midline of the leg.

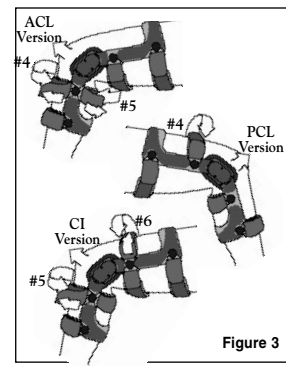
Figure 3

F For ACL version next, secure the upper calf strap (#5). Be careful not to pull this strap too tight and risk circulation problems.

G When all strapping is complete, the hinge should center just above the top of the kneecap and slightly behind midline.

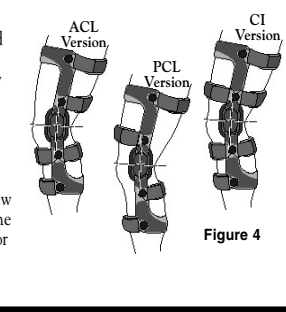
Figure 4

To ensure that no slippage occurs during activity, it is recommended to do a few knee bends or walk for a few minutes, then re-tighten the straps following steps B-F or B-E for CI version.

INSTRUCTIONS FOR ADJUSTABLE LOADING HINGE

Your brace frame has a base corrective load built into it. You may "fine tune" your base load by doing the following: Turn the adjustment screws clockwise using the enclosed Allen wrench tool. You may vary the front and back screw adjustment depending on its interface with the knee and comfort of load. DO NOT turn any 1 screw more than 4 revolutions without toggling back to the other adjustable screw. Be careful not to bind the adjustment mechanism. Do not exceed 10 full revolutions or the pad may become dislodged. If this occurs, reverse the process. Always load brace according to your comfort and activity level.

NOTE: Even if you don't feel immediate relief, it doesn't mean the brace isn't working. It can take several weeks for soft tissue in the knee to adjust to the increased load provided by the brace.

INSTRUCTIONS FOR CHANGEABLE STOPS

Standard Hinge - Your brace comes with 10° extension stops installed. To change the installed stops, first completely remove the screw as shown. Then remove the stop. Insert the new stop, align with the holes in the hinge plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. (Fig. 5)

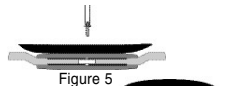
Figure 5

Adjustable Hinge - To change your stops on the adjustable loading hinge, you must first remove the adjustable loading condyle from the hinge by toggling back and forth turning the screws clockwise until the pad is removed. Remove the stop and insert the new stop. To replace the telescoping pad, align the screws with the holes of the hinge plate. While holding it firmly, use the Allen wrench to turn counterclockwise until the threads engage. Toggle back and forth between screws until the pad is returned to its original position. (Fig. 6)

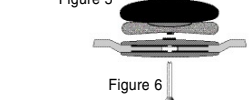
Figure 6

Both medial and lateral stops must be identical in degree. Extension Stops: 0°, 10°, 20°, 30°, and 40°. Flexion Stops: 0°, 45°, 60°, 75°, and 90° (no flexion stops are installed upon shipment)

USE AND CARE

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- All straps and pads may be trimmed to fit individual variations in limb circumference.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners and condyle pads in cold water with mild detergent. AIR DRY ONLY, DO NOT HEAT DRY LINERS OR PADS.
- The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon™ spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water.
- Periodically inspect hinge screws for tightness and retighten as needed.

Always consult with your physician or therapist before making changes to the brace.

Warranty: Congratulations on your purchase of a DonJoy® brand brace. Please be assured that it is made of the finest material and technology available. Your brace comes with a 6 month warranty against defects on soft goods associated with the brace. The Defiance® Adjustable OA brace has a limited lifetime warranty against defects on material and workmanship for the brace frame and hinges. Replacement straps, liners, etc., can be purchased from your local representative, medical provider, or by calling Customer Care at (800) 336-6569, fax (800) 936-6569. The warranty, expressed and implied, shall not apply to any brace which has been repaired and/or serviced by anyone other than dj Orthopedics personnel or by a person authorized to perform such activities by dj Orthopedics. Any modification of brace will result in termination of warranty.

WARNING: THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: Please be advised that dj Orthopedics, LLC does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (ie . . . skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing. Consult your licensed healthcare professional for your activity limitations.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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