ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS: For additional support and symptomatic pain relief of patellofemoral dysfunction including lateral mal-tracking, subluxation/dislocation, and patellar tendonitis.

CONTRAINDICATIONS: Medial subluxation/dislocation.

WARNINGS AND PRECAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Check product for wear prior to application. Do not use if excessive wear is present. Do not use this product in a toxic or corrosive environment. Regular cleaning of this product is recommended.

APPLICATION INFORMATION:

- 1) PARTS: Tru-Pull Advanced Attachment, U-shaped buttress, Double sided patch, (2) Buttons, Adhesive patch.
- 2a) REMOVE: Remove lateral (the outside of brace) condyle pad.
- 2b) THIGH AND CALF FRAME LINER: Peel back thigh and calf frame liner on inside of brace's medial side. **Note: Medial Jog (bend on the inside of the brace).**
 APPLY: Peel off backing and apply button to frame approximately half way between the straps at thigh and calf sections on medial side of brace. **Note: Button to face outside of frame.** See Figure 3
- 3) (2) BUTTONS: Apply buttons to the brace wrapping button flaps tight around the frame to meet or overlap on the inside of the brace under the liner as shown in Figure 3.
 RE-ATTACH: Place the thigh and calf liners back in place.
Note: If necessary apply additional adhesive patch to hold liner in place
- 4) Slide Tru-Pull Advanced Attachment over the plastic condyle shell (from back of the brace toward the front) where the condyle pad was removed.
- 5) RE-APPLY: Place condyle pad using double sided patch to secure it to the Tru-Pull Advanced Attachment as shown in Figure 5. **Note: Various thicknesses of condyle pads can be used to improve comfort and fit.**
 APPLY: U-shaped buttress to the inside of Tru-Pull Advanced Attachment as shown to fit snugly against the lateral side of the patella.
- 6) APPLY: Adjust button hole slots to frame buttons to the desired tension. Trim excess material from the ends of the straps.
 To trim strap ends cut with scissors inside the grooves.

CLEANING INSTRUCTIONS: Detach buttress from brace. Hand wash with mild soap in lukewarm water. Rinse both thoroughly. AIR DRY separately. Note: If not rinsed thoroughly, residual soap can cause irritation to skin or deteriorate material of the product.

WARRANTY: dj Orthopedics, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

For Single Patient Use Only.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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